vivamel
protect

Navodilo za uporabo
VIVAMEL – PROTECT – POLIURETANSKA PENA Z
MEDICINSKIM KOSTANJEVIM MEDOM

Pred prvo uporabo izdelka Vivamel protect – poliuretanska pena z medicinskim kostanjevim medom natančno preberite navodilo. Navodilo shranite. Morda ga boste želeli ponovno prebrati.

KAJ JE VIVAMEL PROTECT – POLIURETANSKA PENA Z
MEDICINSKIM KOSTANJEVIM MEDOM?

Vivamel protect – poliuretanska pena z medicinskim kostanjevim medom je sterilna obloga, narejena iz poliuretanske pene, ki je na eni strani prepojena z medicinskim kostanjevim medom. Poliuretanska pena je dobro vpojna, mehka in izredno prilagodljiva. Zagotavlja dober stik z rano in preprečuje sprijemanje z dnom rane. Na voljo je v dimenziji: 10 x 10 cm, na katero je naneseno 15 g medicinskega kostanjevega medu.

Medicinski med

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1304

vivamel
protect

Instructions for use
VIVAMEL PROTECT – POLYURETHANE FOAM WITH MEDICAL
CHESTNUT HONEY

Before first use of Vivamel protect – polyurethane foam with chestnut honey read the instructions carefully. Keep the instructions for future use.

WHAT IS VIVAMEL PROTECT – POLYURETHANE FOAM WITH
MEDICAL CHESTNUT HONEY?

Vivamel Protect polyurethane foam with medical chestnut honey is a sterile wound dressing made from polyurethane foam and impregnated with medical chestnut honey on one side. Polyurethane foam is highly absorbent, soft, and extremely flexible. It provides good contact with the wound and prevents the dressing from sticking to the wound bed. Available in size 10 x 10 cm with 15 g of medical chestnut honey.

Medical honey

Medical chestnut honey has a high sugar content (up to 80 %), low water content (up to 17 %), and an acidic pH. Due to its physical properties, it has a high osmotic activity, which creates an unfavourable environment for the growth of microorganisms, stimulates wound cleaning, and promotes wound healing. Honey enables moist healing and forms mechanical protection on the wound.

FOR WHICH TYPES OF WOUNDS IS VIVAMEL PROTECT
POLYURETHANE FOAM WITH MEDICAL CHESTNUT HONEY
RECOMMENDED?

Vivamel Protect polyurethane foam with medical chestnut honey is intended for treating and healing a wide spectrum of wounds – chronic and acute ones. It is recommended for treating wounds with moderate to low exudate, especially sensitive wounds in the proliferative phase. Osmotic activity of honey provides bacteriostatic effect and stimulates wound healing.

ADVANTAGES OF USING VIVAMEL PROTECT – POLYURETHANE
FOAM WITH MEDICAL CHESTNUT HONEY AND ITS EFFECT:

- enables moist wound healing
- lowers risk of wound infection
- reduces unpleasant odours
- stimulates necrotic tissue cleansing
- stimulates granulation and epithelisation
- promotes wound healing and prevents scarring
- no sticking onto the wound
- does not cause pain at dressing change
- easy to use
- no damage on the area surrounding the wound

HOW TO USE VIVAMEL PROTECT – POLYURETHANE FOAM WITH
MEDICAL CHESTNUT HONEY?

Apply the side of the dressing that is impregnated with honey and is therefore amber in colour directly onto the wound. Take the wound dressing out of the sterile pouch together with the protective foil. Remove the foil and place the side with honey onto the wound. The dressing should cover the wound completely and may extend over its edges. Vivamel Protect can be cut to fit to the wound.

1. First, rinse the wound with lukewarm running drinking water or saline.
2. Gently dry up the wound and its surrounding area.
3. Take the dressing with protective transparent foil out of the sterile pouch. Remove the foil from one side of the dressing and place the dressing onto the wound, sterile side with no foil on the wound. When placing it, make sure that the dressing covers the whole wound. It can be cut or fold and it may stretch over the wound margins.
4. Remove the protective foil from the other side of the dressing.
5. Protect the dressing by placing a secondary absorbent dressing of your choice over it. In case of smaller amounts of secretion, nonwoven compresses may be used. For wounds with larger amounts of secretion, use a highly absorbent dressing or polyurethane foam. Fasten securely with a fastening bandage, preventing the dressing from moving.

For easier application of Vivamel protect – polyurethane foam with medical chestnut honey use sterile medical devices, e.g. sterile tweezers or tongs. The dressing can be cut in smaller pieces with sterile scissors while still protected by the protective transparent foil.

HOW OFTEN SHOULD ONE CHANGE VIVAMEL PROTECT –
POLYURETHANE FOAM WITH MEDICAL CHESTNUT HONEY?

At the beginning of the healing process, it is recommended that the dressing be changed every two days, in case of substantial amount of secretion even every day, for the effects of honey are decreased by dissolving in wound secretion. Wounds with very little secretion require changing only twice a week. Every time before changing a dressing, remove any remains of previous dressing and dispose of it as infected waste. Clean and dry

up the wound and place a new dressing.

POSSIBLE SIDE EFFECTS

Due to honey's high osmotic activity, which is based on driving water molecules out of the wound, a burning sensation or a slight pain can occur after applying the dressing on the wound, but it usually goes away after 30 minutes to 2 hours. In case of unbearable pain, stop using Vivamel protect – polyurethane foam with medical chestnut honey. Should any unknown side effects occur during the use of the product, ask your doctor or pharmacist or consult the manufacturer or distributor.

WARNINGS

Do not use the dressing if you are allergic to honey or have already experienced a hypersensitivity reaction to honey in the past. Do not use the dressing if the protective pouch has been damaged or opened.

Do not use the dressing if the welds of the packaging are not sealed. Once the wound dressing has been opened, never sterilise and reuse it.

Before and after the application of the dressing, always wash your hands. Avoid contact with eyes and mucosa. In case of accidental contact with eyes or mucosa, rinse with abundant quantities of water. For single use only. Never reuse a used dressing. A used dressing does not contain honey or only a small amount of it, which decreases its effect. Absorbency of a used dressing is also reduced. Reusing a dressing can lead to contamination of the product or the wound, possibly even an infection.

One dressing should never be used on two patients, in order to prevent contamination of the product or the wound, possibly even an infection. Clinical studies of wound healing with Vivamel medical honey in patients with diabetes show very positive results, however, the treatment must be performed under a physician's supervision. Shelf life of the dressing is three years from the production date. After the expiration date, do not use the dressing and dispose of it as normal waste. After four weeks of therapy with Vivamel protect – polyurethane foam, a doctor should review the treatment.

STORING VIVAMEL PROTECT – POLYURETHANE FOAM WITH
MEDICAL CHESTNUT HONEY

Store product in a dry place at temperatures between 5 and 35 °C horizontally, otherwise honey will start leaking from the dressing. Avoid exposing the product to direct sunlight.

STORING VIVAMEL PROTECT – POLYURETHANE FOAM WITH
MEDICAL CHESTNUT HONEY

Store product in a dry place at temperatures between 5 and 35 °C horizontally, otherwise honey will start leaking from the dressing. Avoid exposing the product to direct sunlight.

STORING VIVAMEL PROTECT – POLYURETHANE FOAM WITH
MEDICAL CHESTNUT HONEY

Store product in a dry place at temperatures between 5 and 35 °C horizontally, otherwise honey will start leaking from the dressing. Avoid exposing the product to direct sunlight.

1. First, rinse the wound with lukewarm running drinking water or saline.
2. Gently dry up the wound and its surrounding area.
3. Take the dressing with protective transparent foil out of the sterile pouch. Remove the foil from one side of the dressing and place the dressing onto the wound, sterile side with no foil on the wound. When placing it, make sure that the dressing covers the whole wound. It can be cut or fold and it may stretch over the wound margins.
4. Remove the protective foil from the other side of the dressing.
5. Protect the dressing by placing a secondary absorbent dressing of your choice over it. In case of smaller amounts of secretion, nonwoven compresses may be used. For wounds with larger amounts of secretion, use a highly absorbent dressing or polyurethane foam. Fasten securely with a fastening bandage, preventing the dressing from moving.

REF 19735
VIVAMEL PROTECT – POLYURETHANE FOAM
WITH MEDICAL CHESTNUT HONEY 10 x 10 cm

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Symbol legend: Issue date: September 2019

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